MARCUS & MARCUS®

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EN INSTRUCTION FOR USE:

1. Wash hands thoroughly with soap and clean water before touching the parts. Dry hands with a fresh towel. Avoid touching inside of the parts.
2. Insert connector onto silicone bottle, then place sealing ring on the top.

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
EN TO STORE BREASTMILK

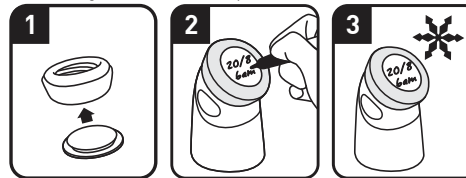1. Attach sealing disc to the collar to form the lid for silicone bottle.
2. Date the expressed milk on the erasable sealing disc with marker.
3. Store in the refrigerator or freezer immediately.

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EN TO PUMP BREASTMILK

1. Attach shield to the collar and screw it onto the silicone bottle.
2. Flip back the shield after assembled.
3. Squeeze the center of the bottle to initiate suction. Place your nipple in the center of shield. Make sure that the silicone bottle is facing downwards.
4. Fold the shield up and release suction for a tight seal. Breastmilk will be expressed by the natural suction created when your let-down begins. Ensure the pump is attached securely before leaving your hands.
5. Once the silicone bottle is full or when you would like to stop, release gently by squeezing the shield area while securely holding the bottle.
6. If the expressed breastmilk is not fed to the baby right away, cover the bottle with sealing disc or pour into Marcus & Marcus Breastmilk Storage bag. Freeze immediately.

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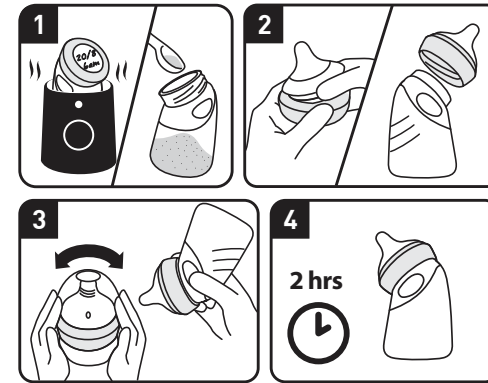
EN TO USE THE FEEDING BOTTLE

1. In order to preserve the components of breastmilk, thaw the frozen breastmilk in the refrigerator overnight or by holding the bottle under running warm water (max 37°C / 98.6°F). Place the thawed breastmilk into a cup, jug or bowl of lukewarm water for a few minutes to bring it to body temperature. Alternatively, use a bottle warmer. Do not allow the temperature to go above 42°C / 108°F. For formula milk, mix powdered formula to hot water over 70°C / 158°F to ensure any bacteria in the formula milk is killed.
2. Attach feeding nipple to the collar with bottle tongs. Avoid touching the nipple with hands. Then, remove the bottle cover and screw the nipple parts on bottle.
3. If milk fat has separated, cover the cap and gently swirl before feeding. Avoid shaking the bottle. Be sure to tighten the collar snugly before feeding. Always check temperature before feeding. Hold the connector and do not squeeze the bottle while feeding.
4. Once breast milk is brought to room temperature or warmed after storing in the refrigerator or freezer, it must be used within 2 hours.

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